



MR. Dental Supplies Limited

4 MANOR WAY, OLD WOKING, SURREY GU22 9JX. ENGLAND
Tel: +44(0)1483 773282 Fax: +44(0)1483 740548 Email: info@mr dental.co.uk Website: www. mrdental.co.uk

INSTRUCTIONS FOR USE

COLD CURE DENTINE KIT:

KITS: 6 or 8 x 30gm coldcure dentine (tooth shades) + 4 Shaders + 120ml Liquid

USES:

Making temporary crowns, bridges and inlays in the dental laboratory. (This material is not suitable for use in the oral cavity.)

MIXING:

The powder and liquid are mixed to a creamy consistency and is ready for use when the material drips smoothly off the end of the spatula. Care should be taken when mixing to ensure that no air is trapped.

CURING:

After the restoration has been "built up" it may either be left to cure atmospherically or in a "pressure pot".

ALL OTHER INSTRUCTIONS AS PER RAPID REPAIR MATERIAL

Liquid Used: Rapid Repair Liquid

WARNING: IN EXTREME RARE CASES MAY CAUSE SENSITISATION BY SKIN CONTACT

USES: Class 1, Temporary Crown & Bridge Material



Issue No: 2
Date: Nov 2019