



MR. Dental Supplies Limited

4 MANOR WAY, OLD WOKING, SURREY GU22 9JX. ENGLAND
Tel:+44(0)1483 773282 Fax:+44(0)1483 740548 Email: info@mrdental.co.uk Website: www.mrdental.co.uk

INSTRUCTIONS FOR USE

MEADWAY DENSACRYL:

Densacryl is a cold curing acrylic which has been specifically formulated for fixing acrylic facing to metal backings without the metal showing through.

METHOD:

1. Select the required colour
2. Roughen the facing, cutting suitable undercuts
3. Mix the tip and body colours in two separate mixing pots with cold cure monomer
4. The material should be applied to the facing as soon as possible and allowed to 'dough'
5. The facing may then be pressed onto the backing into the correct position
6. For the best results, cure in a 'Hydroflask', 'Hi-Lo', or similar pressure pot

In order to prevent metal showing through acrylic, a thin layer of pink can be coated onto the metal before the case is processed.

Densacryl is manufactured in the following shades:

G=Grey
OY=Orange/Yellow
G=Grey/Yellow
P=Pink
Y=Yellow
YG=Yellow/Grey
W=White

The Kit comprises the first six colours plus liquid.

Liquid Used: Meadway Rapid Repair Liquid

WARNING: IN EXTREME RARE CASES MAY CAUSE SENSITISATION BY SKIN CONTACT

USES: MASKING OUT METAL WIRES



Issue No: 2
Date: Nov 2019