



MR. Dental Supplies Limited

4 MANOR WAY, OLD WOKING, SURREY GU22 9JX. ENGLAND
Tel: +44(0)1483 773282 Fax: +44(0)1483 740548 Email: info@mr dental.co.uk Website: www. mr dental.co.uk

MEADWAY HYDROCURE COLDCURE POWDER

- NON SLUMPING
- QUICK SETTING
- NO "PRESSURE POT" NEEDED FOR HARD DENSE REPAIRS, ADDITIONS OR RELINES.
- MATCHES MEADWAY DENTURE ABSE COLOURS
- AVAILABLE IN STANDARD, STANDARD VEINED, TRANSLUCENT AND TRANSLUCENT VEINED COLOURS.

INSTRUCTIONS

Preparation

The broken denture addition or reline is prepared in the usual way.

Mixing

The Powder and Liquid are mixed to a creamy consistency (1.3gm Powder to 1.0ml Liquid)

Care should be taken when mixing to ensure that no air is trapped.

Application

After approximately 30 seconds, the mix is ready for use and a thin coat should be applied over all the prepared surfaces and into the cracks. The material will then begin to reach the 'non-slumping' stage so that the remainder of the repair/addition can be built up.

Care must be taken to ensure that no air is trapped during these operations.

Curing

It is not necessary to use a 'pressure pot' to cure this material and perfect results will be produced provided no air has been trapped during the mixing or building-up stages, although 'pot' could be used if desired to speed-up the process.

The setting time in atmospheric conditions is 15-17 minutes at 21 degree C.

USE WITH MEADWAY COLD CURE RAPID REPAIR LIQUID

USES: DENTURE REPAIRS AND DENTURES



Issue No: 2
Date: Nov 2019