



MR. Dental Supplies Limited

4 MANOR WAY, OLD WOKING, SURREY GU22 9JX. ENGLAND
Tel:+44(0)1483 773282 Fax:+44(0)1483 740548 Email: info@mrdental.co.uk Website: www. mrdental.co.uk

INSTRUCTIONS FOR MEADWAY PLASTER DIVIDER (SEPARATOR)

This product is used where plaster is to be separated from plaster or stone.
Ideal for use in the investment process.

SUGGESTED METHOD OF USE:

- Use the solution directly from the can. **DO NOT DILUTE.**
- Dip the first half of the model into the solution.
- Wipe excess solution from the teeth with cotton wool.
- Allow the model to dry, approximately 2 minutes.
- Invert the second half of the model and flask.
- Process as usual.

USES: PLASTER SEPARATOR / DIVIDER



Issue No: 2
Date: Nov 2019