



MR. Dental Supplies Limited

4 MANOR WAY, OLD WOKING, SURREY GU22 9JX. ENGLAND
Tel:+44(0)1483 773282 Fax:+44(0)1483 740548 Email: info@mrdental.co.uk Website: www. mrdental.co.uk

INSTRUCTIONS FOR USE NAIL ACRYLIC

CREATING A FALSE NAIL:

1. Ensure the surface area of the false hand is dry and free from oil or grease. (You may require a primer to adhere acrylic to silicone).
2. Place the nail form on the finger as shown in diagrams 1 and 2. This form dictates the shape of the final nail.
3. Dispense small amounts of powder and liquid into separate dappen dishes.
4. Take an artists sable brush medium sized, dip the tip into the liquid, then dip the tip only into the powder, with a gentle motion rotate until a small pearl like shape forms on the tip of the brush. (From this point you have approximately 3 mins working time at 20 Deg.C)
5. Gently transfer the material to a position just forward of where the half moon area on the nail would be. With the side of the brush, gently pat the material with a slight pulling motion thus moving the material to the tip. Now push the material forward towards the cuticle area, the material should feather into the cuticle, as the thinner you can make it the better it will adhere and create a more pleasing shape. After creating the cuticle shape return to patting the material out towards the tip all the time reinforcing the lateral curvature. NOTE try to achieve as perfect a nail shape as possible, as this reduces the amount of filing needed to create the finished nail. Before the nail dries, finish off by brushing across the nail, as this greatly adds to its strength.
6. When the nail is completely dried (approximately 6 mins @ 20Deg. C) remove the nail form. You can now file and buff the nail to create a good shape and finish.

USES: FABRICATION OF FALSE NAILS

DIAGRAM 1

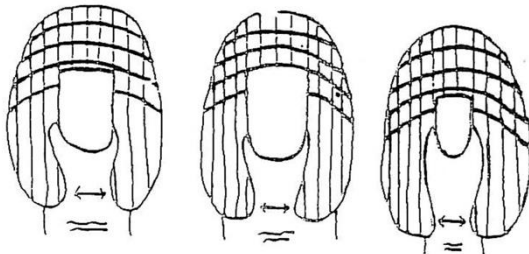


DIAGRAM 2



Issue No: 2
Date: Nov 2019

