



# *MR. Dental Supplies Limited*

4 MANOR WAY, OLD WOKING, SURREY GU22 9JX. ENGLAND  
Tel:+44(0)1483 773282 Fax:+44(0)1483 740548 Email: info@mrdental.co.uk Website: www. mrdental.co.uk

## INSTRUCTIONS FOR MEADWAY TRAY CLEANER AND PLASTER REMOVER

### IMPRESSION TRAYS

Excess plaster and Alginate should be removed from the trays. Immerse them in the Tray Cleaner and Plaster Remover, for approximately one hour. The trays are then rinsed in clean water.

### DENTURES

Meadway Tray Cleaner and Plaster Remover aids the removal of plaster and stone from between teeth and fitted surface of dentures, Dentures should be left in the solution for as long as possible, preferably overnight. Meadway Tray Cleaner and Plaster Remover may also be used to remove plaster and stone in Ultrasonic cleaners.

### WARNING

**MEADWAY TRAY CLEANER AND PLASTER REMOVER IS UNSUITABLE FOR CLEANING ALUMINIUM TRAYS AS SOME CORROSION MAY OCCUR.**

USES: TRAY CLEANER / PLASTER REMOVER



Issue No: 2  
Date: Nov 2019

Manufacturers of Meadway, Medi-Lux and V-Lux Dental Materials & Products  
Also Manufacturers of Tooth Making Equipment  
Registered in England No. 3299060