



MR. Dental Supplies Limited

4 MANOR WAY, OLD WOKING, SURREY GU22 9JX. ENGLAND
Tel:+44(0)1483 773282 Fax:+44(0)1483 740548 Email: info@mrdental.co.uk Website: www. mrdental.co.uk

INSTRUCTIONS FOR MEADWAY WAX SOLVENT

WARNING: HIGHLY FLAMMABLE

CONTAINS ETHER KEEP AWAY FROM ALL SOURCES OF IGNITION
DIRECTIONS FOR USE:

- Before investing immerse cotton pellet in wax solvent.
- Swab off excess wax from teeth.
- Smooth over waxed-up model with cotton pellet.
- Rinse model with soapy water ready for investing.
- After investing pour a small quantity of wax solvent into the flask.
- After the boiling-out process brush around the necks of the teeth to ensure complete removal of wax.
- Flush with clean boiling water to remove all traces of solvent

USES: DISOLVES WAX / WAX REMOVER



Issue No: 2
Date: Nov 2019