



MR. Dental Supplies Limited

4 MANOR WAY, OLD WOKING, SURREY GU22 9JX. ENGLAND
Tel: +44(0)1483 773282 Fax: +44(0)1483 740548 Email: info@mr dental.co.uk Website: www. mr dental.co.uk

INSTRUCTIONS FOR USE

MEDI-LUX AND V-LUX ACRYLIC TEETH:

USES:

In the fabrication of partial or full arch dentures and in the fabrication of temporary space maintainers/ temporary bridges.

PREPARATION:

Select the desired shade and mould.

The teeth are set in the wax pattern of the intended prosthesis. This is then "flasked" and the wax is then "boiled-out" in the usual way.

PACKING OF MATERIAL:

Before packing the material carefully coat the plaster mould with plaster coating solution/ cold mold seal, making sure that the teeth are not coated.

Mix the acrylic / liquid by following the mixing instructions.

Pack the acrylic dough into the plaster mould and cover with a polythene sheet. Replace the flask lid and apply pressure slowly under the bench press. Remove the excess flash of the material.

CURING:

Cure the material according to the curing instructions of the acrylic/ liquid used. Once curing cycle is complete open the flask and remove polythene sheet.

FINISHING:

The prosthesis is trimmed and polished in the usual way.

NB: INTENDED FOR USE IN THE DENTISTRY BY TRAINED / QUALIFIED PROFESSIONAL

USES: FALSE DENTURES



Issue No: 2
Date: Nov 2019