



# *MR. Dental Supplies Limited*

4 MANOR WAY, OLD WOKING, SURREY GU22 9JX. ENGLAND  
Tel: +44(0)1483 773282 Fax: +44(0)1483 740548 Email: info@mr dental.co.uk Website: www. mr dental.co.uk

## INSTRUCTIONS FOR USE

### MODELLING ACRYLIC:

Modelling Acrylic is a Self-Curing Polymer for Various Applications in the Plastic, Dental, Film Industry.

### MIXING:

The powder and liquid are mixed to a creamy consistency (1.3 grams powder to 1.0 ml of liquid)

Care should be taken when mixing to ensure that no air is trapped.

### CURING:

It is not necessary to use a "pressure pot" to cure this material, and perfect results will be produced provided no air has been trapped during the mixing or building up stages, although a "pot" could be used if desired, to speed up the process.

The setting time in atmospheric conditions is 15-17 mins at 21 degrees C.

All other instructions are as per Meadway Rapid Repair and Meadway Premier Rapid Repair.

Liquid Used: Modelling Liquid/ Rapid Repair Liquid

**WARNING: IN EXTREME RARE CASES MAY CAUSE SENSITISATION BY SKIN CONTACT**

**USES:** Main uses are in the special effect departments of the film industry in the fabrication of Teeth, Fangs, Limbs, Joints et.



Issue No: 2  
Date: Nov 2019