



MR. Dental Supplies Limited

4 MANOR WAY, OLD WOKING, SURREY GU22 9JX. ENGLAND
Tel:+44(0)1483 773282 Fax:+44(0)1483 740548 Email: info@mrdental.co.uk Website: www.mrdental.co.uk

INSTRUCTIONS FOR USE

MEADWAY ORTHODONTIC POWDER:

Meadway Orthodontic Resin is a self-curing acrylic resin for the construction of removable orthodontic appliances and temporary dentures.

Method:

1. Soak the model in warm water until the models surface is free from any bubbles.
2. Ensure that the wire work and teeth are fixed firmly.
3. Blow off excess moisture from the surface of the model.
4. Drip 2-4 drops of Monomer onto surface of the model (hold as level as possible). Dispense uniform spray of Polymer onto the Monomer until all the powder particles are saturated. Repeat this procedure until the desire thickness is achieved.

NOTE: No excess Monomer should be left on the surface of the build up prior to curing.

Curing:

For the best results, we recommend the appliance is cured in a pressure flask containing water at 40-50C 2 bar (30psi) for approximately 10 minutes.

5. Trim and polish as usual.

Liquid Used: Meadway Orthodontic Liquid

WARNING: IN EXTREME RARE CASES MAY CAUSE SENTISATION BY SKIN CONTACT

USES: REMOVABLE ORTHODONTIC APPLIANCES



Issue No: 2
Date: Nov 2019