



MR. Dental Supplies Limited

4 MANOR WAY, OLD WOKING, SURREY GU22 9JX. ENGLAND
Tel:+44(0)1483 773282 Fax:+44(0)1483 740548 Email: info@mrdental.co.uk Website: www. mrdental.co.uk

INSTRUCTIONS FOR USE

MEADWAY RAPID REPAIR MATERIAL:

Meadway Rapid Repair is a non - slumping, quick setting, coldcure system and no "PRESSURE POT" is needed for hard dense repairs or additions or relines.

AVAILABLE IN STANDARD, TRANSLUCENT, AMERICAN, DEEP PINK, LIGHT PINK SHADES WITH OR WITHOUT VEINS

Preparation:

The broken denture addition or reline is prepared in the usual way.

Mixing:

The Powder and Liquid are mixed to a creamy consistency (1.3gm Powder to 1.0ml Liquid)

Care should be taken when mixing to ensure that no air is trapped.

Application:

After approximately 30 seconds, the mix is ready for use and a thin coat should be applied over all the prepared surfaces and into the cracks. The material will then begin to reach the 'non-slumping' stage so that the remainder of the repair/addition can be built up.

Care must be taken to ensure that no air is trapped during these operations.

Curing:

It is not necessary to use a 'pressure pot' to cure this material and perfect results will be produced provided no air has been trapped during the mixing or building-up stages, although 'pot' could be used if desired to speed-up the process.

The setting time in atmospheric conditions is 15-17 minutes at 21 degree C.

Liquid Used: Rapid Repair Liquid

WARNING: IN EXTREME RARE CASES MAY CAUSE SENSITISATION BY SKIN CONTACT

USES: DENTURE REPAIRS & DENTURES



Issue No: 2
Date: Nov 2019

Manufacturers of Meadway, Medi-Lux and V-Lux Dental Materials & Products
Also Manufacturers of Tooth Making Equipment
Registered in England No. 3299060