



MR. Dental Supplies Limited

4 MANOR WAY, OLD WOKING, SURREY GU22 9JX. ENGLAND
Tel: +44(0)1483 773282 Fax: +44(0)1483 740548 Email: info@mr dental.co.uk Website: www. mrdental.co.uk

INSTRUCTIONS FOR USE

MEADWAY COLD CURE DENTINE:

USES:

Making temporary crowns, bridges and inlays in the dental laboratory.

(This material is not suitable for use in the oral cavity)

MIXING:

The Powder and Liquid are mixed to a creamy consistency and is ready for use when the material drips smoothly off the end of the spatula.

Mixing Ratios:

1.3 grams powder to 1.0ml of liquid

Care should be taken when mixing to ensure that no air is trapped.

CURING:

After the restoration has been "built up" it may either be left to cure atmospherically or in a "pressure pot". Although a "pot" could be used if desired to speed up the process.

The setting time in atmospheric conditions is 15-17 mins at 21 degrees c.

Liquid used: Meadway Rapid Repair Liquid.

WARNING: IN EXTREME RARE CASES MAY CAUSE SENSITISATION BY SKIN CONTACT

USES: Class 1, Temporary Crown & Bridge Material



Issue No: 2
Date: Nov 2019