



MR. Dental Supplies Limited

4 MANOR WAY, OLD WOKING, SURREY GU22 9JX. ENGLAND
Tel:+44(0)1483 773282 Fax:+44(0)1483 740548 Email: info@mrdental.co.uk Website: www.mrdental.co.uk

MEADWAY DENSACRYL

Densacryl is a cold curing acrylic which has been specifically formulated for fixing acrylic facing to metal backings without the metal showing through.

INSTRUCTIONS

1. Select the required colour.
2. Roughen the facing, cutting suitable undercuts.
3. Mix the tip and body colours in two separate mixing pots with cold cure monomer.
4. The material should be applied to the facing as soon as possible and allowed to 'dough'.
5. The facing may then be pressed onto the backing into the correct position.
6. For the best results, cure in a 'Hydroflask', 'Ho-lo', or similar pressure pot.

In order to prevent metal showing through acrylic, a thin layer of pink can be coated onto the metal before the case is processed.

Densacryl is manufactured in the following shades:

G=Grey
OY=Orange/Yellow
G=Grey/Yellow
P=Pink
Y=Yellow
YG=Yellow/Grey
W=White

The Kit comprises the first six colours plus liquid.

Liquid Used: Meadway Rapid Repair Liquid

WARNINGS AND CAUTIONS:

- **IN EXTREME RARE CASES MAY CAUSE SENSITISATION BY SKIN CONTACT**
- **USE ONLY BY TRAINED PROFESSIONALS**
- **IT IS THE RESPONSIBILITY OF THE TRAINED PROFESSIONAL TO ENSURE THAT THE PRODUCT IS SUITABLE FOR END USER**
- **STORE BETWEEN 16C TO 23C AWAY FROM DIRECT SUNLIGHT**
- **DISPOSABLE OF UNCURED LIQUID MUST BE CARRIED OUT IN ACCORDANCE WITH MATERIAL SAFETY DATA SHEET**
- **AND MIXED POWDER AND LIQUID CAUSES EXOTHERMIC REACTION**
- **IF THE DEVICE HAS BEEN PLACED IN THE ORAL CAVITY THEN IT MUST BE DISPOSED OF AS POTENTIALLY BIOLOGICAL HAZARD WASTE**
- **THE USER AND/OR THE PATIENT MUST REPORT ANY SERIOUS INCIDENT THAT HAS OCCURRED IN RELATION TO THE DEVICE TO THE MANUFACTURER AND THE COMPETENT AUTHORITY OF THE MEMBER STATE IN WHICH THE USER AND/OR PATIENT IS ESTABLISHED**



Issue No: ORIGINAL

Date: 21ST May 2021

Manufacturers of Meadway, Medi-Lux and V-Lux Dental Materials & Products
Also Manufacturers of Tooth Making Equipment
Registered in England No. 3299060