

INSTRUCTIONS FOR MEADWAY WAX SOLVENT

WARNING: HIGHLY FLAMMABLE - CONTAINS ETHER KEEP AWAY FROM ALL SOURCES OF IGNITION

DIRECTIONS FOR USE:

- Before investing immerse cotton pellet in wax solvent
- Swab off excess wax from teeth
- Smooth over waxed-up model with cotton pellet
- Rinse model with soapy water ready for investing
- After investing pour a small quantity of wax solvent into the flask
- After the boiling-out process brush around the necks of the teeth to ensure complete removal of wax
- Flush with clean boiling water to remove all traces of solvent

Manufactured by:
MR. Dental Supplies Limited
4 Manor Way
Old Woking
Surrey, England
GU22 9JX

Telephone: 00 44 (0) 1483 773282
Fax: 00 44 (0) 1483 740548



Issue No:

Date:

Signed By:

Approved By: