



MR. Dental Supplies Limited

4 MANOR WAY, OLD WOKING, SURREY GU22 9JX. ENGLAND
Tel: +44(0)1483 773282 Fax: +44(0)1483 740548 Email: info@mrdental.co.uk Website: www. mrdental.co.uk

INSTRUCTIONS FOR USE

COLD CURE DENTINE KIT:

KITS: 6 or 8 x 30gm Coldcure dentine (tooth shades) + 4 Shaders + 120ml Liquid

USES:

Making temporary crowns, bridges and inlays in the dental laboratory. (This material is not suitable for use in the oral cavity.)

MIXING:

The powder and liquid are mixed to a creamy consistency and is ready for use when the material drips smoothly off the end of the spatula. Care should be taken when mixing to ensure that no air is trapped.

CURING:

After the restoration has been "built up" it may either be left to cure atmospherically or in a "pressure pot".

ALL OTHER INSTRUCTIONS AS PER RAPID REPAIR MATERIAL

Liquid Used: Rapid Repair Liquid

WARNING: IN EXTREME RARE CASES MAY CAUSE SENSITISATION BY SKIN CONTACT

USES: Class 1, Temporary Crown & Bridge Material

WARNINGS AND CAUTIONS:

- *IN EXTREME RARE CASES MAY CAUSE SENSITISATION BY SKIN CONTACT*
- *USE ONLY BY TRAINED PROFESSIONALS*
- *IT IS THE RESPONSIBILITY OF THE TRAINED PROFESSIONAL TO ENSURE THAT THE PRODUCT IS SUITABLE FOR END USER*
- *STORE BETWEEN 16C TO 23C AWAY FROM DIRECT SUNLIGHT*
- *DISPOSABLE OF UNCURED LIQUID MUST BE CARRIED OUT IN ACCORDANCE WITH MATERIAL SAFETY DATA SHEET*
- *AND MIXED POWDER AND LIQUID CAUSES EXOTHERMIC REACTION*
- *IF THE DEVICE HAS BEEN PLACED IN THE ORAL CAVITY THEN IT MUST BE DISPOSED OF AS POTENTIALLY BIOLOGICAL HAZARD WASTE*
- *THE USER AND/OR THE PATIENT MUST REPORT ANY SERIOUS INCIDENT THAT HAS OCCURRED IN RELATION TO THE DEVICE TO THE MANUFACTURER AND THE COMPETENT AUTHORITY OF THE MEMBER STATE IN WHICH THE USER AND/OR PATIENT IS ESTABLISHED*

Issue No: ORIGINAL

Date: 21ST May 2021



Manufacturers of Meadway, Medi-Lux and V-Lux Dental Materials & Products
Also Manufacturers of Tooth Making Equipment
Registered in England No. 3299060