



MR. Dental Supplies Limited

4 MANOR WAY, OLD WOKING, SURREY GU22 9JX. ENGLAND
Tel: +44(0)1483 773282 Fax: +44(0)1483 740548 Email: info@mr dental.co.uk Website: www. mr dental.co.uk

INSTRUCTIONS FOR USE

MEDI-LUX AND V-LUX ACRYLIC TEETH:

USES:

In the fabrication of partial or full arch dentures and in the fabrication of temporary space maintainers/ temporary bridges.

PREPARATION:

Select the desired shade and mould.

The teeth are set in the wax pattern of the intended prosthesis. This is then "flasked" and the wax is then "boiled-out" in the usual way.

PACKING OF MATERIAL:

Before packing the material carefully coat the plaster mould with plaster coating solution/ cold mold seal, making sure that the teeth are not coated.

Mix the acrylic / liquid by following the mixing instructions.

Pack the acrylic dough into the plaster mould and cover with a polythene sheet. Replace the flask lid and apply pressure slowly under the bench press. Remove the excess flash of the material.

CURING:

Cure the material according to the curing instructions of the acrylic/ liquid used. Once curing cycle is complete open the flask and remove polythene sheet.

FINISHING:

The prosthesis is trimmed and polished in the usual way.

USES: FALSE DENTURES

WARNINGS AND CAUTIONS:

- *IN EXTREME RARE CASES MAY CAUSE SENSITISATION BY SKIN CONTACT*
- *USE ONLY BY TRAINED PROFESSIONALS*
- *IT IS THE RESPONSIBILITY OF THE TRAINED PROFESSIONAL TO ENSURE THAT THE PRODUCT IS SUITABLE FOR END USER*
- *STORE BETWEEN 16C TO 23C AWAY FROM DIRECT SUNLIGHT*
- *DISPOSABLE OF UNCURED LIQUID MUST BE CARRIED OUT IN ACCORDANCE WITH MATERIAL SAFETY DATA SHEET*
- *AND MIXED POWDER AND LIQUID CAUSES EXOTHERMIC REACTION*
- *IF THE DEVICE HAS BEEN PLACED IN THE ORAL CAVITY THEN IT MUST BE DISPOSED OF AS POTENTIALLY BIOLOGICAL HAZARD WASTE*
- *THE USER AND/OR THE PATIENT MUST REPORT ANY SERIOUS INCIDENT THAT HAS OCCURRED IN RELATION TO THE DEVICE TO THE MANUFACTURER AND THE COMPETENT AUTHORITY OF THE MEMBER STATE IN WHICH THE USER AND/OR PATIENT IS ESTABLISHED*



Issue No: ORIGINAL

Date: 21ST May 2021

Manufacturers of Meadway, Medi-Lux and V-Lux Dental Materials & Products
Also Manufacturers of Tooth Making Equipment
Registered in England No. 3299060