



# MR. Dental Supplies Limited

4 MANOR WAY, OLD WOKING, SURREY GU22 9JX. ENGLAND  
Tel: +44(0)1483 773282 Fax: +44(0)1483 740548 Email: info@mr dental.co.uk Website: www. mr dental.co.uk

MEADWAY PLASTER COATING SOLUTION (PCS):

## MIXING DEHYDRATED PLASTER COATING SOLUTION

### TO MAKE 5LTS

1. Place 5lts De-Ionised water in large clean polythene container.
2. Gradually mix in 114gm Powder and 100ml Dye/Solution.
3. Keeping water well agitated.
4. Mix again at frequent intervals, stirring the mixture 3 or 4 times.
5. Continue this process until all the constituents have dissolved.
6. Allow to stand for 24 hours. The material will then be ready to use.

### WARNINGS AND CAUTIONS:

- *IN EXTREME RARE CASES MAY CAUSE SENSITISATION BY SKIN CONTACT*
- *USE ONLY BY TRAINED PROFESSIONALS*
- *IT IS THE RESPONSIBILITY OF THE TRAINED PROFESSIONAL TO ENSURE THAT THE PRODUCT IS SUITABLE FOR END USER*
- *STORE BETWEEN 16C TO 23C AWAY FROM DIRECT SUNLIGHT*
- *DISPOSABLE OF UNCURED LIQUID MUST BE CARRIED OUT IN ACCORDANCE WITH MATERIAL SAFETY DATA SHEET*
- *AND MIXED POWDER AND LIQUID CAUSES EXOTHERMIC REACTION*
- *IF THE DEVICE HAS BEEN PLACED IN THE ORAL CAVITY THEN IT MUST BE DISPOSED OF AS POTENTIALLY BIOLOGICAL HAZARD WASTE*
- *THE USER AND/OR THE PATIENT MUST REPORT ANY SERIOUS INCIDENT THAT HAS OCCURRED IN RELATION TO THE DEVICE TO THE MANUFACTURER AND THE COMPETENT AUTHORITY OF THE MEMBER STATE IN WHICH THE USER AND/OR PATIENT IS ESTABLISHED*



Issue No: ORIGINAL

Date: 21<sup>ST</sup> May 2021