



MR. Dental Supplies Limited

4 MANOR WAY, OLD WOKING, SURREY GU22 9JX. ENGLAND
Tel: +44(0)1483 773282 Fax: +44(0)1483 740548 Email: info@mr dental.co.uk Website: www. mr dental.co.uk

INSTRUCTIONS FOR MEADWAY TRAY CLEANER AND PLASTER REMOVER

IMPRESSION TRAYS

Excess plaster and Alginate should be removed from the trays. Immerse them in the Tray Cleaner and Plaster Remover, for approximately one hour. The trays are then rinsed in clean water.

DENTURES

Meadway Tray Cleaner and Plaster Remover aids the removal of plaster and stone from between teeth and fitted surface of dentures, Dentures should be left in the solution for as long as possible, preferably overnight. Meadway Tray Cleaner and Plaster Remover may also be used to remove plaster and stone in Ultrasonic cleaners.

WARNINGS AND CAUTIONS:

- *IN EXTREME RARE CASES MAY CAUSE SENSITISATION BY SKIN CONTACT*
- *USE ONLY BY TRAINED PROFESSIONALS*
- *IT IS THE RESPONSIBILITY OF THE TRAINED PROFESSIONAL TO ENSURE THAT THE PRODUCT IS SUITABLE FOR END USER*
- *STORE BETWEEN 16C TO 23C AWAY FROM DIRECT SUNLIGHT*
- *DISPOSABLE OF UNCURED LIQUID MUST BE CARRIED OUT IN ACCORDANCE WITH MATERIAL SAFETY DATA SHEET*
- *AND MIXED POWDER AND LIQUID CAUSES EXOTHERMIC REACTION*
- *IF THE DEVICE HAS BEEN PLACED IN THE ORAL CAVITY THEN IT MUST BE DISPOSED OF AS POTENTIALLY BIOLOGICAL HAZARD WASTE*
- *THE USER AND/OR THE PATIENT MUST REPORT ANY SERIOUS INCIDENT THAT HAS OCCURRED IN RELATION TO THE DEVICE TO THE MANUFACTURER AND THE COMPETENT AUTHORITY OF THE MEMBER STATE IN WHICH THE USER AND/OR PATIENT IS ESTABLISHED*
- *MEADWAY TRAY CLEANER AND PLASTER REMOVER IS UNSUITABLE FOR CLEANING ALUMINIUM TRAYS AS SOME CORROSION MAY OCCUR.*



Issue No: ORIGINAL

Date: 21ST May 2021